

# CATCH-ALL™

## CARE AND MAINTENANCE GUIDELINES

[www.HartwellMedical.com](http://www.HartwellMedical.com)

*Creating Quality Products for Life*

### CARE AND MAINTENANCE

The CATCH-ALL™ provides an easy and convenient storage area for small medical equipment that is frequently used on a majority of EMS calls. The flexible, vinyl-coated material is lightweight, durable and made of waterproof material that will not absorb blood or body fluids. These Care and Maintenance Guidelines are recommendations for maximizing the life and service of this product. You should always follow the specific cleaning and disinfecting practices dictated by your Medical Director.

---

### CLEANING PROCEDURE

The CATCH-ALL should be cleaned using soap and water, a mild detergent, or a commercial cleaner/disinfectant. Any blood, body fluids or other soiled materials should be thoroughly removed prior to disinfection. Sodium hypochlorite (bleach) solutions may be used for disinfection, but avoid prolonged exposure of the vinyl material to high concentrations of bleach to avoid discoloration. A 1% bleach solution (or concentration compatible with current CDC Guidelines) can be used on the CATCH-ALL, but it should be thoroughly rinsed off with cold water and then allowed to dry. Other commercially available disinfecting agents suitable for use on vinyl surfaces may be wiped or sprayed onto the CATCH-ALL following the product manufacturer's recommended instructions for use, and allowed to dry completely.

---

### CUSTOMER SERVICE

Hartwell Medical is known for its outstanding customer service and warranty program. We deliver high-quality, innovative emergency medical products and stand behind each one. Should you have any questions about the use or care of any of our products, our customer service personnel will be happy to help you.

#### **Customer Service Hours:**

Monday - Friday  
8:00am - 4:30pm, PST  
(760) 438-5500  
(800) 633-5900  
[customerservice@hartwellmedical.com](mailto:customerservice@hartwellmedical.com)

Additional information on our entire product line is available on our website:

[www.HartwellMedical.com](http://www.HartwellMedical.com)

### ABOUT US

Hartwell Medical is proud to have been in business for over 20 years as a highly respected manufacturer of innovative emergency medical products. Over the past two decades we have seen significant technological advances in the emergency care industry and are proud to have played a key role in the research and development of improved patient care equipment and techniques.

All of our products are subjected to clinical evaluation and real-life field tests to ensure they meet our high standards and your ever-changing needs. Should you have any questions about the use or care of our products, we will be happy to help. Hartwell Medical is continually working to provide you with quality products for life.

## **HARTWELL MEDICAL**

6354 Corte del Abeto, Suite F, Carlsbad, California 92011

760-438-5500

800-633-5900

760-438-2783 fax