

CombiCarrierII® Head Immobilizer

APPLICATION GUIDELINES

www.HartwellMedical.com
Creating Quality Products for Life

INTRODUCTION

The unique design of the CombiCarrierII® Head Immobilizer allows the Base Plate to be secured to the CombiCarrierII® and left on the device in the ambulance storage compartment. The Head Immobilizer is x-ray compatible, easily cleaned, compact and simple to use!

INCLUDED IN THE PACKAGE

The CombiCarrierII® Head Immobilizer consists of the following:

1 each	Base Plate
2 each	Support Blocks
1 each	Head Strap

APPLICATION INSTRUCTIONS

The photos to the right show how the Head Immobilizer is easily attached to the CombiCarrierII® and used to secure a patient.

Remember to always follow your local medical protocols established by your medical director.

1. Attach the Base Plate of the Immobilizer to the head of the CombiCarrierII® by fastening the side straps around the back of the CombiCarrierII® and the top strap through the hole at the top of the CombiCarrierII®.
2. Front of the CombiCarrierII® with Base Plate attached.
3. Place the Head Support Blocks on both sides of the patient's head, making sure the blocks fit snugly against the patient's head, just above their ears.
4. Secure the patient's head with the Head Strap, passing each end through the D-rings on either side of the Base Plate and securing the strap with the hook and loop material on the ends of each strap.

1.



2.



3.



4.



HARTWELL MEDICAL

CombiCarrierII® Head Immobilizer

APPLICATION GUIDELINES

www.HartwellMedical.com
Creating Quality Products for Life

ORDERING INFORMATION

MODEL

CC 3200H

DESCRIPTION

CombiCarrierII® Head Immobilizer

REPLACEMENT PARTS

CC 3200-9-100

CC 3200-9-200

CC 3200-9-300

DESCRIPTION

CombiCarrierII® Head Immobilizer Base Plate (each)

CombiCarrierII® Head Immobilizer Head Support Block (pair)

CombiCarrierII® Head Immobilizer Head Strap (each)

CUSTOMER SERVICE

Hartwell Medical is known for its outstanding customer service and warranty program. We deliver high-quality, innovative emergency medical products and stand behind each one. Should you have any questions about the use or care of the CombiCarrierII® Head Immobilizer, our customer service personnel will be happy to help you.

Please retain this sheet for reference should you need to order more product or replacement parts in the future. This information will assist our customer service department in obtaining the proper parts for you.

Customer Service Hours:

Monday - Friday

8:00am - 4:30pm, PST

(760) 438-5500

(800) 633-5900

customerservice@hartwellmedical.com

Additional information on Hartwell Medical and the products we offer is always available on our website:

www.HartwellMedical.com

ABOUT US

Hartwell Medical is proud to have been in business for over 20 years as a highly respected manufacturer of innovative emergency medical products. Over the past two decades we have seen significant technological advances in the emergency care industry and are proud to have played a key role in the research and development of improved patient care equipment and techniques.

It all started in 1989 with a new splint designed to treat extremity injuries using vacuum technology instead of air pressure. Today, Hartwell Medical manufactures the two best-selling vacuum splint products in the industry; the well known EVAC-U-SPLINT® and the new, semi-disposable FASPLINT®. In 1999, Hartwell Medical introduced the award-winning CombiCarrier®, the first combination backboard and scoop stretcher. This innovative device saved rescuers valuable budget dollars and storage space by combining 2 products into 1. Today, the next generation CombiCarrierII® is even thinner, lighter, and stronger than the original and continues to be the ideal patient handling and transfer device.

All of our products are subjected to clinical evaluation and real-life field tests to ensure they meet our high standards and your ever-changing needs. Should you have any questions about the use or care of our products, we will be happy to help. Hartwell Medical is continually working to provide you with quality products for life.

HARTWELL MEDICAL

6354 Corte del Abeto, Suite F, Carlsbad, California 92011

760-438-5500

800-633-5900

760-438-2783 fax