

# CombiCarrierII® Head Immobilizer

## APPLICATION GUIDELINES

[www.HartwellMedical.com](http://www.HartwellMedical.com)  
Creating Quality Products for Life

### INTRODUCTION

The unique design of the CombiCarrierII® Head Immobilizer allows the Base Plate to be secured to the CombiCarrierII and left on the device in the ambulance storage compartment. The Head Immobilizer is x-ray compatible, easily cleaned, compact and simple to use! These instructions are applicable to both the CC 3400H and the CC 3400H-BK.

---

### INCLUDED IN THE PACKAGE

The CombiCarrierII Head Immobilizer consists of the following:

1 each	Base Plate
2 each	Support Blocks
1 each	Head Strap

---

### APPLICATION INSTRUCTIONS

The photos to the right show how the Head Immobilizer is easily attached to the CombiCarrierII and used to secure a patient.

**IMPORTANT: Always follow your local medical protocols established by your medical director.**

1. Attach the Base Plate of the Immobilizer to the head of the CombiCarrierII by threading the long strap through the “D” shaped holes of the CombiCarrierII as shown in photo 1. The strap is threaded up through the hole, across the top face of the CombiCarrierII, back down through the hole on the other side and then secures to the loop material on the top of the Base Plate.
2. Front of the CombiCarrierII with Base Plate attached.
3. Once the patient’s body has been properly secured, place the Head Support Blocks on both sides of the patient’s head, just above their ears, making sure the blocks fit snugly against the patient’s head. The base plate strap end will not hinder the retention of the Head Support Block once the Head Strap is in place.
4. Secure the patient’s head with the Head Strap, passing each end through the D-rings on either side of the Base Plate and securing the strap with the hook and loop material on both ends of the strap.



# CombiCarrierII® Head Immobilizer

## APPLICATION GUIDELINES

[www.HartwellMedical.com](http://www.HartwellMedical.com)  
Creating Quality Products for Life

### ORDERING INFORMATION

#### MODEL #

CC 3400H  
CC 3400H-BK

#### DESCRIPTION

CombiCarrierII® Head Immobilizer  
CombiCarrierII Head Immobilizer - Black

#### REPLACEMENT PARTS

CC 3400-9-100  
CC 3400-9-200  
CC 3400-9-300  
CC 3400-9-100-BK  
CC 3400-9-200-BK  
CC 3400-9-300-BK

#### DESCRIPTION

CombiCarrierII Head Immobilizer Base Plate (each)  
CombiCarrierII Head Immobilizer Head Support Block (pair)  
CombiCarrierII Head Immobilizer Head Strap (each)  
CombiCarrierII Head Immobilizer Base Plate - Black (each)  
CombiCarrierII Head Immobilizer Head Support Block - Black (pair)  
CombiCarrierII Head Immobilizer Head Strap - Black (each)

### CUSTOMER SERVICE

Hartwell Medical is known for its outstanding customer service and warranty program. We deliver high-quality, innovative emergency medical products and stand behind each one. Should you have any questions about the use or care of the CombiCarrierII® Head Immobilizer, our customer service personnel will be happy to help you.

If you become aware of any serious incident related to the use of the device, you should report the incident to Hartwell Medical and to your local medical device regulatory authority.

Please retain this sheet for reference should you need to order more product or replacement parts in the future. This information will assist our customer service department in obtaining the proper parts for you.

#### **Customer Service Hours:**

Monday - Friday  
8:00 am - 4:30 pm, PST  
[customerservice@hartwellmedical.com](mailto:customerservice@hartwellmedical.com)

### ABOUT US

Hartwell Medical is proud to have been in business for more than 30 years as a highly respected manufacturer of innovative emergency medical products. Over the past two decades we have seen significant technological advances in the emergency care industry and are proud to have played a key role in the research and development of improved patient care equipment and techniques.

All of our products are subjected to clinical evaluation and real-life field tests to ensure they meet our high standards and your ever-changing needs. Should you have any questions about the use or care of our products, we will be happy to help. Hartwell Medical is continually working to provide you with quality products for life.

Additional information on Hartwell Medical and the products we offer is always available on our website.

## HARTWELL MEDICAL



Hartwell Medical LLC  
6354 Corte del Abeto, Suite F  
Carlsbad, CA 92011 USA  
PH: (760) 438-5500 FX: (760) 438-2783  
[www.HartwellMedical.com](http://www.HartwellMedical.com)



Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

