

SUREVENT[®] FlowMaster[™] MCI Kit

CARE AND MAINTENANCE GUIDELINES

www.HartwellMedical.com

Creating Quality Products for Life

CARE AND MAINTENANCE

The SUREVENT[®] FlowMaster[™] MCI Kit is designed for compact storage and easy portability. The rugged manifold is mounted on an aluminum base that stores the 20 ft. oxygen supply hose, all contained in a secure case that leaves plenty of room for storing Emergency Automatic Resuscitators, oxygen masks, cannulas, or other oxygen therapy accessories. These Care and Maintenance Guidelines are recommendations for maximizing the life and service of this product. **You should always follow the specific cleaning and disinfecting practices dictated by your Medical Director.**

CLEANING PROCEDURE

The case itself and the manifold should be cleaned using soap and water, a mild detergent, or a commercial cleaner/disinfectant. It is best to use a damp cloth for cleaning and a separate cloth for rinsing these components. Care should be taken to not allow any liquids to enter the flowmeters or the manifold. Any blood, body fluids or other soiled materials should be thoroughly removed prior to disinfection. Sodium hypochlorite (bleach) solutions may be used for disinfection, but avoid prolonged exposure to high concentrations of bleach to avoid discoloration. A 1% bleach solution (or concentration compatible with current CDC Guidelines) can be used, but it should be thoroughly rinsed off with cold water and then allowed to dry. Other commercially available disinfecting agents suitable for use on these surfaces may be wiped or sprayed onto them following the product manufacturer's recommended instructions for use, and allowed to dry completely.

MAINTENANCE

The SUREVENT FlowMaster MCI Kit is ideal for disaster caches and to have on hand for everyday emergency care because it requires no annual calibration. Its compact design and watertight, protective hard case make storage easy.

CUSTOMER SERVICE

Hartwell Medical is known for its outstanding customer service and warranty program. We deliver high-quality, innovative emergency medical products and stand behind each one. Should you have any questions about the use or care of any of our products, our customer service personnel will be happy to help you.

Customer Service Hours:

Monday - Friday

8:00 am - 4:30 pm, PST

(760) 438-5500

(800) 633-5900

customerservice@hartwellmedical.com

Additional information on our entire product line is available on our website:

www.HartwellMedical.com

ABOUT US

Hartwell Medical is proud to have been in business for more than 25 years as a highly respected manufacturer of innovative emergency medical products. Over the past two decades we have seen significant technological advances in the emergency care industry and are proud to have played a key role in the research and development of improved patient care equipment and techniques.

All of our products are subjected to clinical evaluation and real-life field tests to ensure they meet our high standards and your ever-changing needs. Should you have any questions about the use or care of our products, we will be happy to help. Hartwell Medical is continually working to provide you with quality products for life.

HARTWELL MEDICAL

CMFMCI 09/16

6354 Corte del Abeto, Suite F, Carlsbad, California 92011

760-438-5500

800-633-5900

760-438-2783 fax