

SUREVENT® FLOWMASTER™ MCI KIT

Multi-Patient Ventilation/Oxygenation System



- Complete Solution for Multi-Patient Emergency Respiratory Care
- Essential Disaster Management Component
- Ideal Fire Ground Oxygen Rehabilitation Station
- A Single Staging Point for Up to 7 Patients
- Single Gas Source Connects to Multi-Outlet Manifold
- Manifold Base Stores 20 Feet of Oxygen Hose
- Adjustable Flow Control Outlets Conserve Oxygen and Provide Ultimate Versatility

A Complete Solution to Ventilation and Oxygen Support


Why Use Automated Ventilatory Support?

The SUREVENT® FlowMaster™ MCI Kit offers the ultimate in ease of use, versatility and utility. Matched with Emergency Automatic Resuscitators (not included) the FlowMaster MCI Kit provides a truly complete and cost effective mass casualty automatic ventilation system. Each kit provides a single staging point for up to 7 patients. Individual controls on each of the seven patient oxygen outlets allow flow adjustments from 0 to 25 LPM to accommodate the unique needs of each patient as well as to allow both air-mix and 100% oxygen devices to be connected to the manifold at the same time. An in-line pressure gauge allows the operator to monitor and assure proper oxygen supply working pressure at all times.

Why Use the SUREVENT FlowMaster MCI Kit?

The SUREVENT FlowMaster MCI Kit may also be used as an oxygen therapy station for fireground rehabilitation and multiple patient applications using oxygen masks, cannulas, nebulizers and many other oxygen powered devices. One flow controller is detachable to provide a male DISS check valve outlet for connecting other oxygen powered devices.* The rugged manifold is mounted on an aluminum base that stores the 20 ft. oxygen supply hose, all contained in a secure, waterproof case that leaves plenty of room for storing Emergency Automatic Resuscitators, oxygen masks, cannulas, or other oxygen therapy accessories. A laminated instruction placard with a seven patient management system on the back and a waterproof marking pencil are included, as well as a supplemental laminated flow guide. The case can be stored compactly and is well marked for easy identification.

*Maximum manifold flow limitations (150 LPM at 50 psig) must be observed for device accuracy.

SPECIFICATIONS	KIT CONTENTS - Model SV 1530:	
<p>EXTERIOR DIMENSIONS: 20.7"L x 17.2"W x 8.6"D (52.6cm x 43.7cm x 20.8cm)</p> <p>WEIGHT: 20.4 lbs. (9.3kg)</p> <p>OPERATING ENVIRONMENTAL LIMITS: 0 to 122° F (-18° to 50° C)</p> <p>STORAGE ENVIRONMENTAL LIMITS: -31° to 140° F (-35° to 60° C)</p> <p>GAS SOURCE: 50 psi Source for Connection to Manifold</p>	<p>(1) Hartwell Medical 7 Outlet Oxygen Manifold Base Unit with Pressure Gauge</p> <p>(7) Adjustable Flowmeter Control Outlets</p> <p>(1) 20' Section of Oxygen Hose with DISS Fittings</p> <p>(2) Laminated Instructional Placards</p> <p>(1) Watertight, Hard Case - Color: Olive Drab Green</p>	

HARTWELL MEDICAL

6354 Corte del Abeto, Suite F
 Carlsbad, CA 92011
 800-633-5900
 760-438-5500 Tel
 760-438-2783 Fax
 www.HartwellMedical.com

Authorized Dealer:

Creating Quality Products for Life